



Fine Products Since 1978

# Companion Rollator

INSTRUCTIONS  
& WARRANTY

## **WARNING!**

When using the Transport Rollator, users with limited physical capabilities should be supervised and assisted when necessary.

---

Transport Rollator needs to be properly balanced before use. Special attention should be taken to ensure that height and hand adjustments are secure, and that all components are in good working condition. Make sure that ALL wheels are in contact with the ground at ALL times.

---

Make sure that the brakes are locked BEFORE sitting down on the seat. When sitting down on rollator, DO NOT lean to one side or the other. Be sure to distribute your weight evenly at all times. Do not sit on rollator if ground is sloped or uneven.

---

Press the back of your legs against the seat. Do not use rollator as a wheelchair on rough or uneven surfaces. Doing so may cause it to tip-over, resulting in injury. Never use stairs or escalators while using the rollator. Do not use on inclines over 8%.

# ASSEMBLY

## INSTALL HANDLE BARS

1. Remove adjustment knob and bolt from handles.
2. Insert handles into rollator frame.
3. With your arm relaxed at your side, position handle so grip is level with bend in your wrist.
4. Insert bolt through plastic washer, frame and handle.
5. Attach securely with knob and tighten.

## INSTALL BACKREST

1. Insert backrest into frame until snap buttons engage.
2. Ensure backrest is securely fastened.

## INSTALL BASKET

1. Lift up seat.
2. Place basket under seat, ensuring tabs rest on frame.

## INSTALL PUSH BAR

1. Insert push bar into frames until snap buttons engage.
2. Ensure push bar is securely fastened.

---

## USING YOUR ROLLATOR AS A WALKER

### FOLDING YOUR ROLLATOR

1. Remove push bar.
2. Pull up on the seat.
3. Remove basket from under the seat if installed.
4. Pull up on nylon cord to fold.

### UNFOLDING YOUR ROLLATOR

1. Place all wheels on ground and squeeze brakes.
2. Push down on handles and extend front casters away from you. With one hand, press down on seat until wheels fully extend.
3. Install push bar if required.

### SITTING ON YOUR ROLLATOR

1. Make sure rollator is on a level surface.
2. Lock hand brakes of rollator by pressing down on grips toward the ground until they lock into position.
3. Turn around until your back is to the rollator.
4. Slowly step back until back of your legs touch rollator.

## WALKING WITH YOUR ROLLATOR

1. Make sure foot plates are vertical and out of the way of your feet.
  2. Standing behind rollator, roll the rollator slowly forward so that rear wheels are a few inches in front of your feet.
  3. Step forward and place your foot in line with rear wheels but slightly inside of them.
  4. Maintaining a steady roll forward, place your other foot in line with rear wheels.
  5. Determine a safe pace to keep your feet in line with rear wheels. Do not go on steep slopes or ramps that might cause you to lose control.
- 

## USING YOUR TRANSPORT ROLLATOR AS A TRANSPORT CHAIR

1. For short trips, the transport rollator allows your Caregiver to push you on level surfaces such as hallways, parking lots, malls, airports, and sidewalks.
  2. Have your Caregiver hold push bar while you are getting seated and when standing up.
  3. Fold down foot plates and place both of your feet on the plates.
  4. While being pushed, the patient should have feet on the folded-down foot plates at all times.
  5. Always have your Caregiver lock the brakes when you are not being pushed or ambulating.
  6. Never try to use on steep surfaces or soft terrain.
  7. This product is not intended to be used as a full time wheelchair.
- 

## KEEP THIS FOR YOUR RECORDS

If there are any problems with your rollator, DO NOT attempt to repair it yourself. Contact the dealer/store that you purchased the rollator from for any needed parts or repair.